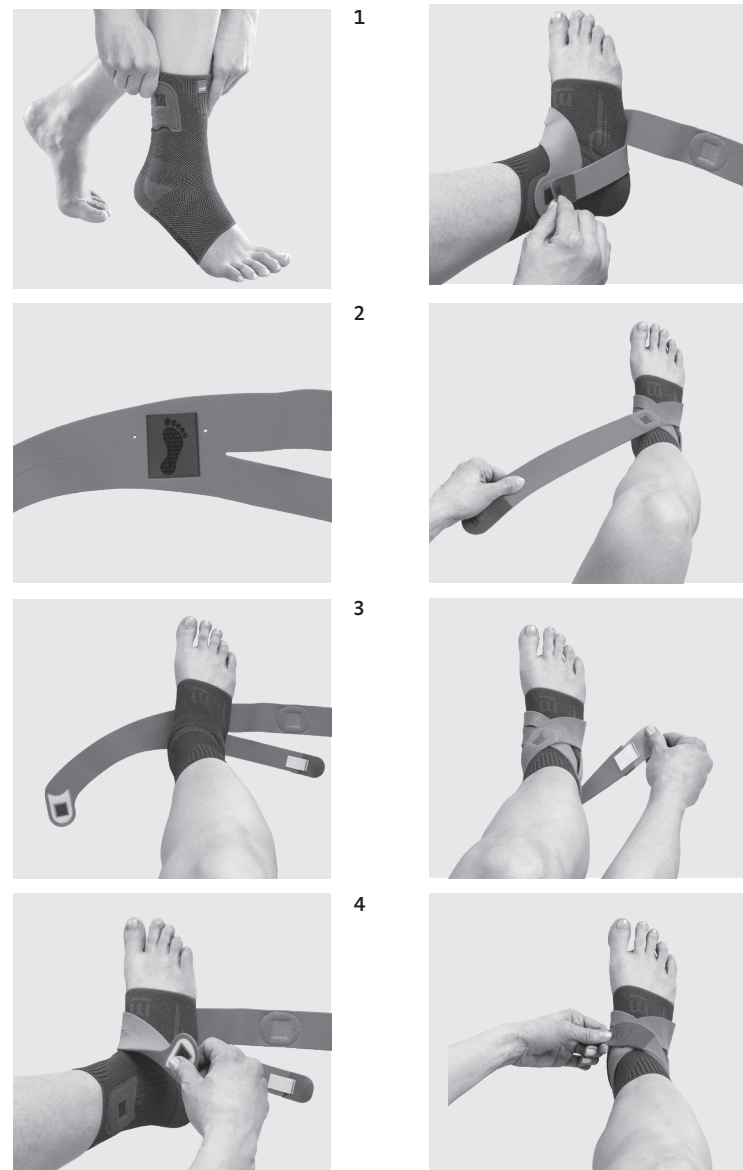
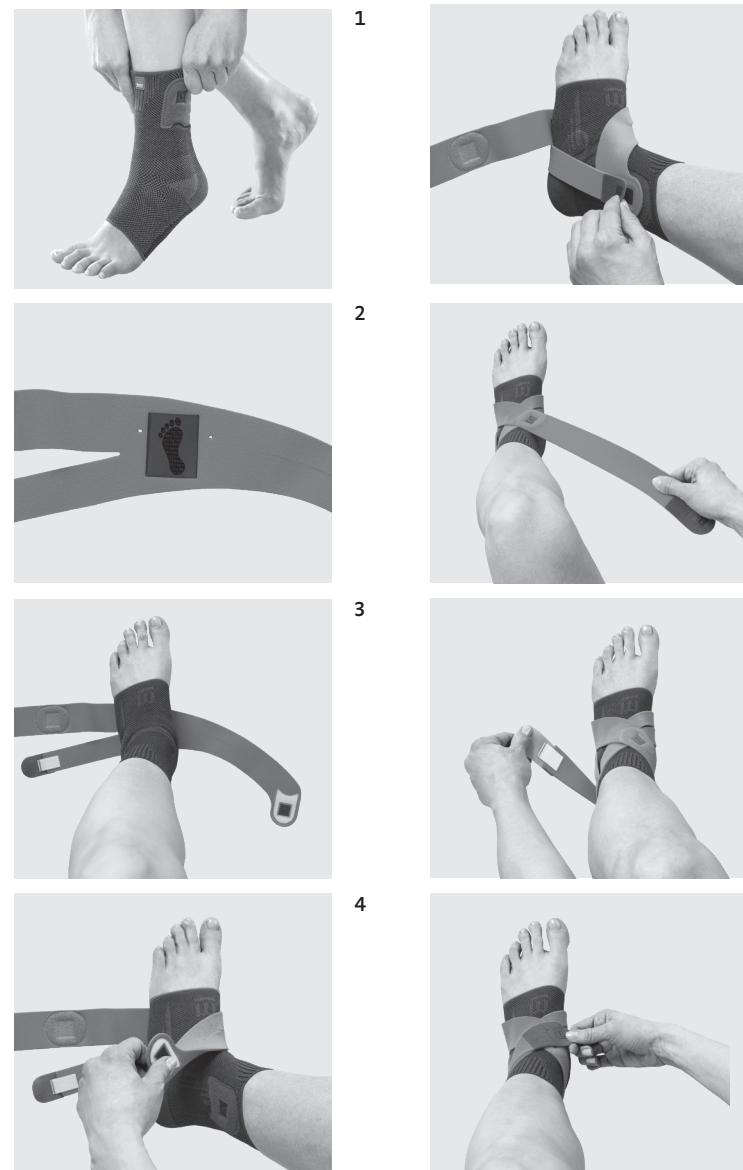
## Rechter Fuß

## Linker Fuß

Deutsch

## Levamed active

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Polyamid, Polyester, Elasthan, Viskose

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# Levamed active

## Intended purpose

Levamed active is a bandage for compression of the soft tissue in the ankle joint with additional functional elements.

## Indications

All indications which call for soft tissue compression with additional local cushion support on the ankle and where movement is limited in at least one direction of mobilisation via additional functional elements, such as:

- Sprains with capsule strain
- Chronic ligament instability with recurrent sprained ankle (twisted ankle)
- In the second treatment phase for upper ankle lateral collateral ligament rupture
- Conservative treatment of syndesmosis ligament injury which does not require surgery

## Contraindications

None known at present.

## Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)

- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

## Intended users and patient target groups

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

## Note on wearing

The brace achieves its greatest effect during physical activity. In general, the brace can be worn all day. However, this should be done according to your comfort. During longer breaks (e.g. prolonged sitting / road trips, sleeping), the brace should be removed.

## Fitting instructions

- Take hold of the top of the bandage on both sides and pull it over your foot until the heel part is sitting correctly

and the silicone inserts are positioned around the ankle. (Fig. 1)

- Place the attached strap on the floor in front of you, with the side with the footprint on it facing up. (Fig. 2)
- Now position your foot on the symbol so that the strap runs under the middle of the sole of your foot. Ensure that your foot is at 90° to your shin, as far as is possible. (Fig. 3)
- Take hold of the end of the strap, which is on the inside of your foot  and wrap it over the upper side of your foot diagonally so that it is now on the outer side of your foot. Fasten the end of the strap above the outer side of your ankle, using the hook and loop area intended for this purpose. (Fig. 4)
- Then take the shorter strap on the outside of your foot  and fasten it above the outer side of your ankle too. (Fig. 5) Pull on this end of the strap to secure the entire belt around your foot. This ensures comfortable stability for the joint.
- Finally, take the longer strap  on the outside of your foot across the top of your foot diagonally and then around the leg, bringing it to the front. Stick the end of the strap to the marking on the strap under it. (Fig. 6+7+8)

NOTE: If you need to, you can shorten the strap to fit you by cutting it at the markings  or . To do this, unfasten the hook and loop fasteners at the end of the strap and shorten it accordingly.

### Care instructions

Hook and loop fasteners should be closed for washing. Fabric softeners, fats, oils, lotions, ointments and soap residues can affect the material, hinder the Clima Comfort effect and cause skin irritations.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

### Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

### Material composition

Polyamide, polyester, elastane, viscose

### Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

### Disposal

The product can be disposed of in the domestic waste.

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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