medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-57
medi@medi.de
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61-2 9890 8696
F +61-2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium NV
Posthoornstraat 13/1
3582 Koersel
Belgium
T: +32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Brasil
Rua Cristovan de Vita, n.º 260, Un. 30,
Centro Logístico Raposo Tavares,
Vila Camargo
Vargem Grande Paulista - SP
CEP 06730-000
Brasil
T: +55-11-3500 8005
sac@medibrasil.com
www.medibrasil.com

medi Canada Inc / médi Canada Inc
597, Rue Duvernay, Verchères
QC Canada J0L 2R0
T: +1 450-583-3317 / +1 800 361 3153
F: +1 888-583-6827
service@medicanada.ca
www.medicanada.ca

MAXIS a.s.,
medi group company
Slezská 2127/13
120 00 Prague 2
Czech Republic
T: +420 571 633 510
F: +420 571 616 271
info@maxis-medica.com
www.maxis-medica.com

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbaek Strand
Denmark
T +45-70 25 56 10
F +45-70 25 56 20
kundeservice@sw.dk
www.medidanmark.dk

medi Bayreuth Espana SL
C/Canigo 2-6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34-932 60 04 00
F +34-932 60 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33-1 48 61 76 10
F +33-1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1 371-0090
F +36 1 371-0091
info@medi.hu
www.medi.hu

medi Italia s.r.l.
Via Giuseppe Ghedini, 2
40069 Zola Predosa (BO)
Italy
T: +39-051 6 13 24 84
F: +39-051 6 13 29 56
info@medi-italia.it
www.medi-italia.it

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31-76 57 22 555
F +31-76 57 22 565
info@medi.nl
www.medi.nl

medi

mediven® ulcer kit

Gebrauchsanweisung. Instructions for use. Mode
d'emploi. Instrucciones de uso. Instruções de uso.
Istruzioni per l'uso. Gebruiksaanwijzing. Bruksanvisning.
Brugsvejledning. Brugsanvisning.

4 046114 001744

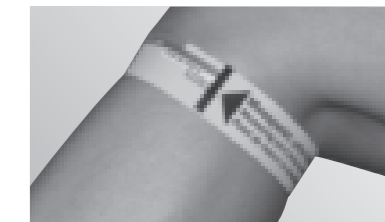
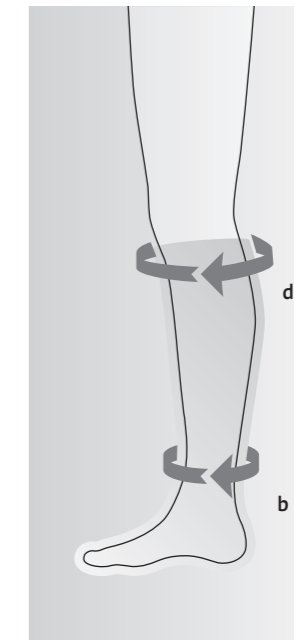
EO05174 / 06.2020

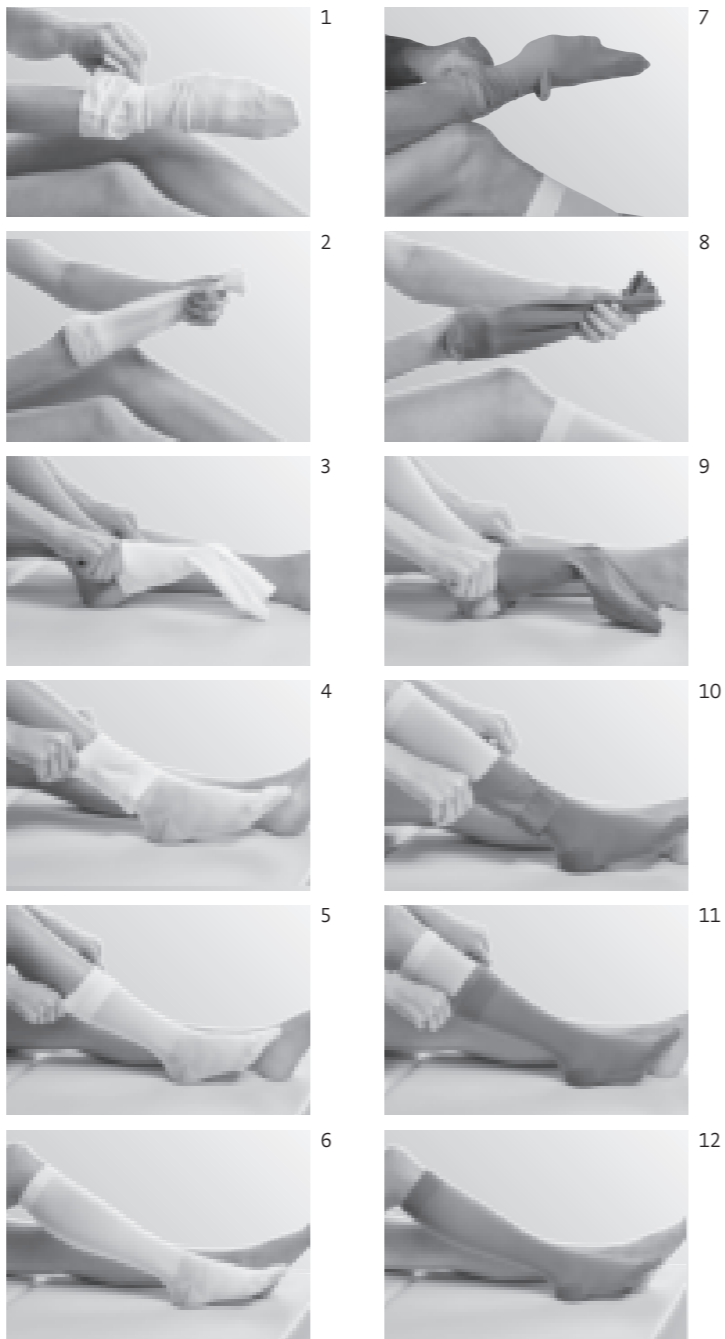
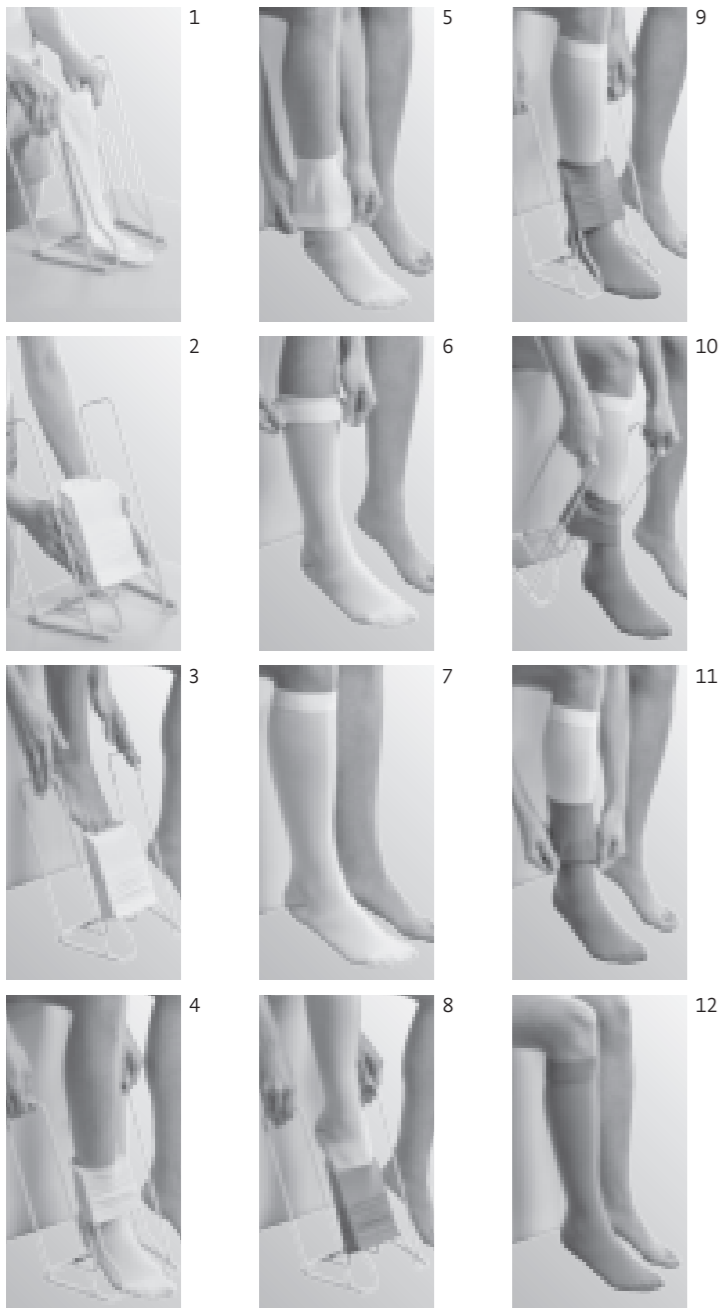
Deutsch	2
English	5
عربي	10
български	11
中文	14
Dansk	16
Français	19
Ελληνικά	22
עברית	27
Italiano	28
日本語	31
Hrvatski	34
Nederlands	37
Norsk	40
Polski	43
Português	46
Română	49
Русский	52
Svenska	55
Slovenský	58
Slovenščina	61
Español	64
Čeština	67
Türkçe	70
Українська	73
Magyar	76
Suomi	79
Eesti	82
Lietuvių	85
Latviešu	88

mediven® ulcer / mediven® ulcer plus

	I	II	III	IV	V	VI	VII
cD	27–33	29–36	32–39	34–42	36–45	38–48	40–50
cC	28–34	30–37	33–40	35–43	37–46	39–49	41–51
cB1	23–27	24–29	26–32	29–35	31–37	33–39	35–41
cB	18–20	20–22	22–24	24–26	26–28	28–30	30–32
cY	26–31	28–33	29–35	31–37	32–38	33–40	34–42
cA	17–22	19–24	21–26	23–29	25–32	27–34	28–35

Alle Angaben in cm	Länge	ID 34–38 kurz	ID 39–44
All measurements are in cm	Length	ID 34–38 petite	ID 39–44
Toutes les mesures sont en cm	Longuer	ID 34–38 court	ID 39–44
Todos los datos se indican en cm	Longitud	ID 34–38 corta	ID 39–44
Todos as medidas em cm	Comprimento	ID 34–38 curto	ID 39–44
Tutti i dati si intendono in cm	Lunghezza	ID 34–38 corto	ID 39–44
Alle maten in cm	Lengte	ID 34–38 kort	ID 39–44
Alle mål er angivet i cm	Længde	ID 34–38 kort	ID 39–44
All mått i cm	Längd	ID 34–38 kort	ID 39–44
Wszystkie wartości podane są w cm	Długość	ID 34–38 krótki	ID 39–44
Усі розміри в сантиметрах	Довжина	ID 34–38 кор.	ID 39–44
Všetchny údaje v cm	délka	ID 34–38 krátká	ID 39–44
Všetky údaje v cm	dĺžka	ID 34–38 krátka	ID 39–44
所有测量值的单位均为厘米	长度	ID 34–38 娇小型	ID 39–44

Deutsch

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Instructions for use

Intended purpose

Circular-knit compression stocking system used for compression of the lower extremities, primarily in the treatment of disorders of the venous system.

Indications

- Therapy for venous ulcers (CEAP classification C6)
- Therapy for mixed ulcers (due to arterial and venous issues), taking the contraindications into account
- Prevention of recurrence of venous ulcer (CEAP classification C5)

Contraindications

- Advanced peripheral arterial occlusive disease (if one of these parameters is present: ABPI < 0.5, ankle arterial pressure < 60 mmHg, toe pressure < 30 mmHg or TcPO₂ < 20 mmHg on dorsum of foot)
- Decompensated heart failure (NYHA III + IV)
- Septic phlebitis
- Phlegmasia cerulea dolens

Particular attention to

- Severe weeping dermatosis
- Intolerance to compressive material
- Severe paraesthesia in the limbs
- Advanced peripheral neuropathy (e.g. as seen in diabetes mellitus)
- Primary chronic polyarthritis

patient target group

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals. Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their

own responsibility.

Appropriate usage

Taking measurements correctly is a prerequisite for enabling the system to do its job optimally. Here, the size of the ankle (cB) dictates the size to be selected. Measure below the knee (cD) to confirm the selected size.

Contact with chemical substances (e.g. benzenes, methanols, salicylic acids in high concentrations) may impair the compressive function of the product. Oils and ointments should not be applied to the limb prior to application of the garment.

If you experience pain or increased skin irritation while wearing the stockings, take off the stocking immediately and consult your doctor or carer.

When a wound is present, always cover with appropriate dressing prior to application of compression. The skin should be inspected on a daily basis.

Only one stocking should be worn if you are immobile for long periods of time.

In order to ensure safe use, the understocking mediven ulcer should be washed at least every 2-3 days.

Content

2 mediven ulcer
1 mediven ulcer plus
1 measuring tape
1 donning aid for mediven ulcer plus open toe

Storage

Keep this product out of direct sunlight
Store in a dry place

Service life / life span

A precisely graduated compression pattern is required to ensure that these stockings work optimally. Daily wearing and washing of the stockings can cause the compression and elasticity of your stockings, which are required to make them medically effective, to deteriorate over time. The stockings should therefore only be worn for a maximum of six months before being replaced. Observe the instructions for putting on the stockings and instructions for care to ensure that the compression pressure is maintained as well as possible.

Disposal

Please dispose of your medical compression stockings via your general waste or household waste.

There are no specific criteria for disposing of the stockings.

This compression product does not contain any latex.

Care instructions

This compressive product can be washed on a delicate cycle and dried in a tumble dryer set to a low heat.

- Delicate wash cycle (mediven ulcer plus)
- Hot cycle up to 95°C (mediven ulcer)
- Do not bleach
- Dry in the tumble dryer at a low heat
- Do not iron
- Do not dry-clean

Wearing recommendation

Unless otherwise specified by the doctor, wear your mediven compression stocking from morning to night for optimal therapeutic results. The under-stocking mediven ulcer can also be worn at night or when you are at rest.

Donning without Butler

Putting on mediven ulcer without

- Gather the stocking or turn the stocking inside out up to the heel.
- First, pull the foot section over the foot until the size marking is positioned over the heel and the dorsum of the foot.
- Now pull the stocking over your ankle and calf until it is under the knee, at which point you can pull the stocking to the desired length. Do not twist the stocking while doing this and ensure that there are no wrinkles in it.
- Then check that it fits correctly and make sure that it is well-positioned and wrinkle-free. Ensure that the colour mark is sitting over the middle of the heel.

Putting on the mediven ulcer plus without the medi Butler

- Gather the stocking or turn the stocking inside out up to the heel
- Pull the foot section over the foot until the size marking is positioned over the heel and the dorsum of the foot.
- Now pull the stocking over your ankle and calf until it is under the knee, at which point you can pull the stocking to the desired length. Do not twist the stocking while doing this and ensure that there are no wrinkles in it.

Ensure that the bands of both stockings are sitting on top of each other and that there are no wrinkles in the toes.

Donning with Butler

Putting on the mediven ulcer with the medi Butler with long handles

- Roll the mediven ulcer over the step-in frame of the medi Butler until the colour mark appears over the edge of the frame.
- Step into the stocking, placing your foot through the stocking and onto the floor until you are standing on the floor.
- Now pull the stocking up to the middle of your calf using the handles. Tilt the medi Butler backwards to remove it.

- Pull the stocking up evenly until it sits below the knee. You can then pull it up further if you wish.
- Then check that it fits correctly and make sure that it is well-positioned and wrinkle-free. Ensure that the colour mark is sitting over the middle of the heel.

Putting on the mediven ulcer plus with the medi Butler

- Roll the mediven ulcer over the step-in frame of the medi Butler until the colour mark appears over the edge of the frame.
- Step into the stocking, placing your foot through the stocking and onto the floor until you are standing on the floor.
- Now pull the stocking up to the middle of your calf using the handles. Tilt the medi Butler backwards to remove it.
- Pull the stocking up evenly until it sits below the knee. You can then pull it up further if you wish.
- Now pull the stocking up until it sits under the knee. You can then pull it up further if you wish.

Ensure that the bands of both stockings are sitting on top of each other and that there are no wrinkles in the toes.

Complaints

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

Declaration of symbols

 Symbol for medical device (Medical Device)

 Symbol for single patient – multiple use → This product is intended to be used multiple times by a single patient

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